

*GP Order of Draw*

**Sarstedt S-Monovette® System**

***TUBE GUIDE FOR GENERAL PRACTITIONERS***

**\*BOTTLES MUST BE USED IN THE FOLLOWING ORDER**

**FAILURE TO COMPLY MAY COMPROMISE PATIENT RESULT & PATIENT SAFETY**



PD-GEN-05  
Edition No. 1.04  
Date: September 2019

Colour Code	Tube Type / Order*	Investigation(s)
 White	Serum <b>1<sup>st</sup></b> <b>STAND UPRIGHT ONCE DRAWN</b>	Serology Hepatitis screening, HIV, Viral screens  Vitamin D, Testosterone, Active B12
 Green	Sodium Citrate Coag <b>2<sup>nd</sup></b>	PT, APTT, Fibrinogen <b>Underfilled tube will be rejected</b>
 Brown	Serum (Gel tube) <b>3<sup>rd</sup></b> <b>STAND UPRIGHT ONCE DRAWN</b>	All Immunology <b>One separate full tube for Immunology</b> LDH, Lithium Drug Levels
 Orange	Lithium Heparin Gel <b>4<sup>th</sup></b>	Routine Biochemistry Profiles Routine Endocrine Tests  <b>One full tube sufficient for Biochemistry and Endocrine Tests</b>
 Grey	EDTA <b>5<sup>th</sup></b>	FBC, IM Screen  <b>Only tube accepted is Sarstedt 05.1167.901 (CS02146 MMUH Euro Route Logistics)</b>
 Pink	EDTA <b>6<sup>th</sup></b>	HbA1c PTH, Red Cell Folate <b>All of the above require a separate tube</b>  <b>Only tube accepted is Sarstedt 04.1917.001 (CS02139 MMUH Euro Route Logistics)</b>
 Yellow	Fluoride EDTA <b>7<sup>th</sup></b>	Glucose, Glucose Tolerance Test (GTT)  <b>Only tube accepted is Sarstedt 04.1918.001 (CS0143 MMUH Euro Route Logistics)</b>
 Purple	ESR <b>8<sup>th</sup></b>	ESR <b>Underfilled tube will be rejected</b>

*Refer to Laboratory for more specialised tests.*

**Please display this beside your venepuncture equipment in your clinical area**

